

Debridement Pathway

Wound Types




Dry sloughy necrotic*



Infected wounds
Consider antibiotic cover



Dry sloughy

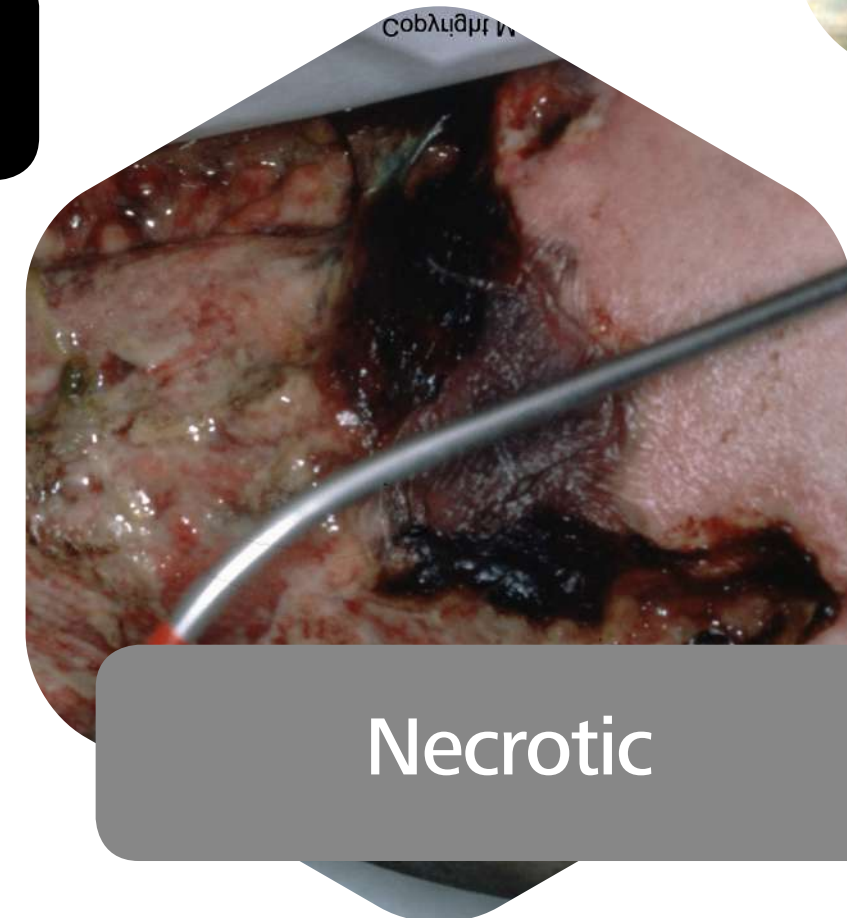

Necrotic



Sloughy



Dressing

HydroClean® plus
Rapid wound debridement
Fast. Simple. Effective^[2,3]

Wear time

Up to 3 days

Review


In line with current hospital wound protocol i.e Weekly assessments of wound

Suitable for wounds with low/no exudate  to moderate exudate 

*if Lower Leg check circulation prior to debridement. Do not debride e.g ischaemic limbs/ high-risk areas.

When to debride

Multidisciplinary team decision pathway^[4]

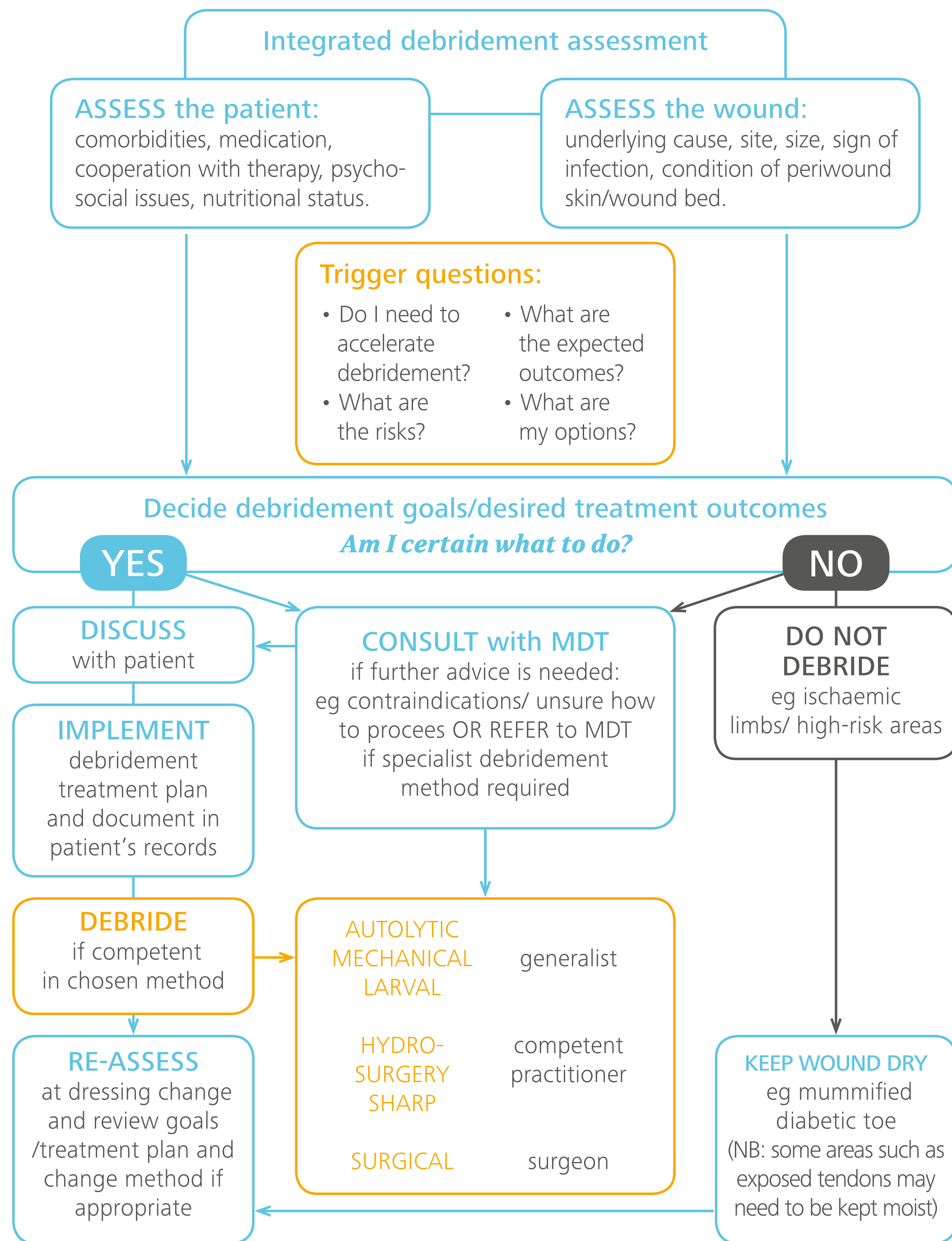

How to fixate HydroClean® plus

For superficial wounds

Use a secondary dressing of your choice i.e film / foam to fixate and cover the HydroClean® plus directly.



1. Always ensure the surrounding skin is cleaned and dry. Lift the protective cover 1.

For deeper wounds

Gently fill the wound bed with HydroClean® plus mini/cavity and cover the wound with a HydroClean® plus. Use a secondary dressing of your choice e.g. Hydrofilm® to fixate the dressing.



2. Place the exposed adhesive onto HydroClean® plus and totally remove the cover 1 while smoothing down the film dressing.



3. Then remove the protective cover 2 and press the adhesive borders onto the skin surrounding HydroClean® plus.



4. Gently remove covers 3 and then 4 using the flap to hang on to and smooth down the film dressing towards the edges.

Suitable for use under compression

[1] Strohler, R., Apelqvist, J., Dissemond, J. et al. EWMA Document: Debridement. J Wound Care. 2013; 22 (Suppl. 1): S1-S52.
 [2] Hodgson, H. et al. (2017). A multicentre, clinical evaluation of a hydro-responsive wound dressing: the Glasgow experience. Journal of Wound Care 26(11), pp. 643-650. [3] Atkin, L. and Ousey, K. (2016). Wound bed preparation: A novel approach using HydroTherapy. British Journal of Community Nursing 21 (Suppl. 12), pp. S23-S28 [4] International expert opinion from: 1. Wounds UK (2013). Effective debridement in a changing NHS. A UK consensus. Available from: www.wounds-uk.com

HydroClean® plus

Product overview

Type	Dimensions	NPC Code	PIP Code
HydroClean® plus	4cm round	ELZ871	401 1540
HydroClean® plus	5.5cm round	ELZ872	401 1557
HydroClean® plus	7.5 x 7.5cm	ELZ873	401 1565
HydroClean® plus	10 x 10cm	ELZ874	401 1573
HydroClean® plus mini	3cm round	ELZ977	401 5581
HydroClean® plus Cavity	4cm round	ELZ869	401 1581
HydroClean® plus Cavity	7.5 x 7.5cm	ELZ870	401 1599

HydroClean® plus



4cm round



5.5cm round



7.5 x 7.5cm



10 x 10cm

HydroClean® plus mini



3cm round

HydroClean® plus Cavity



4cm round



7.5 x 7.5cm

HARTMANN UK & Ireland

Unit P2 Parklands, Heywood Distribution Park
Pilsworth Road, Heywood OL10 2TT

Phone: +44 (0) 1706 363200 | Fax: +44 (0) 1706 363201 | Email: info@uk.hartmann.info

www.hartmann.co.uk

©2019 Paul Hartmann Limited. ® represents a trademark of Paul Hartmann Limited, a company registered in England and Wales (company number 01523121)
WM-00029/19-GB



Going further
for health