

Zetuvit® Plus

Real-life evidence for effective
exudate management

Multi-centre Investigation

A 50 patient observational clinical study of a superabsorbent dressing: effective exudate management with **Zetuvit® Plus** evaluated by clinicians and patients

Zetuvit® Plus – We are all in this together

For 200 years HARTMANN has worked together with Health Care Professionals (HCPs) to find the most effective, simple and convenient wound treatment solutions that benefit both HCPs and their patients. One of the outcomes of the LINK Network is the **Zetuvit® Plus** Multi-centre Investigation



LINK

Learn. Inform. Network. Knowledge.

Design: Open, non-comparative, multi-centre investigation

Number of Patients: 50

Types of Wounds: A variety of different types of wounds – mainly chronic – with medium to high exudate production levels. [e.g. venous leg ulcers (29%), pressure ulcers (22%), diabetic foot ulcers (8%)]

Location: United Kingdom

Objective:

Assess the use of a superabsorbent wound dressing used for exudate management in medium to high exuding wounds




Highly effective exudate management:

In **100% of the cases** Zetuvit® Plus achieved the primary objective, offering **effective exudate management**.

Prevention of leakage and **strikerthrough** were rated “excellent” or “good” in **96%** and **98%**, respectively.^[1]

Supports improved wound healing:

A significant **decrease of patients showing maceration** (65%) was reported. More than half of all patients experienced **improved periwound skin** and overall there was a trend towards **reduction in wound size**.^[1]

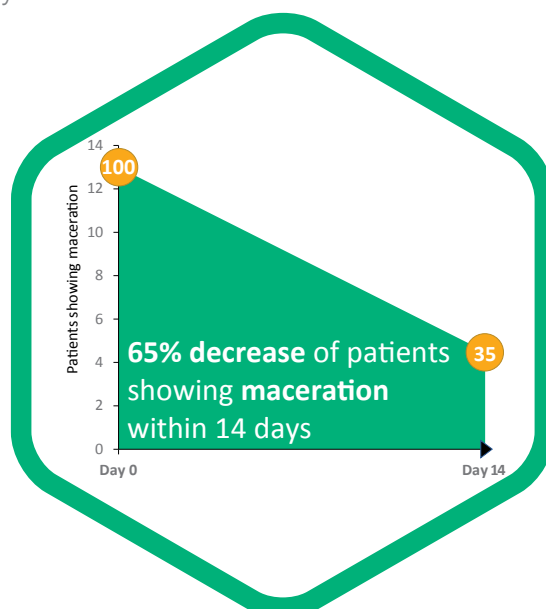


Zetuvit® Plus – effective and efficient exudate management.^[1]

Cost-effective and time saving*:

47% cost savings compared to previous exudate management treatments.

Zetuvit® Plus reduced the **frequency of dressing change from 6 to 2.7** times per week.^[1]

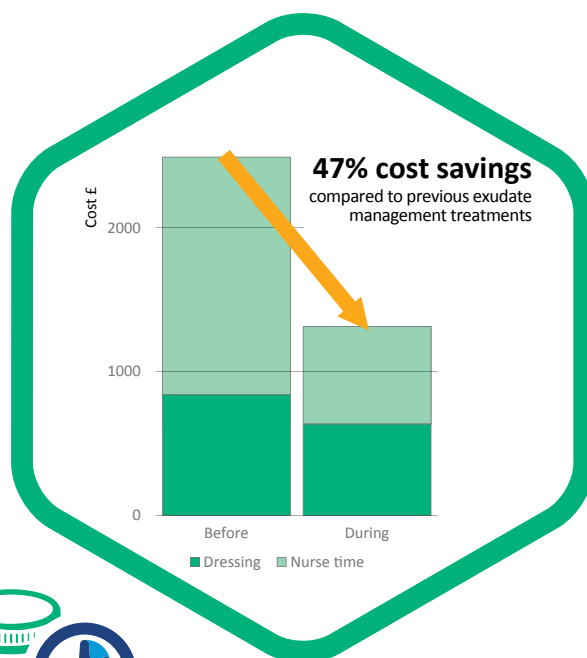

Patients reported they were able to **return to previous activities of daily living** due to the very good exudate handling capabilities and comfort of **Zetuvit® Plus**.

96% rated **patient satisfaction and comfort** "excellent" or "good".^[1]



Going further
for health

100% of clinicians participating in the study
would continue to use **Zetuvit® Plus**.^[1]

Product	Size	Product Code	PIP Code	NCP Code	Pack Size
Zetuvit® Plus	10 x 10 cm	413710	3567351	EME046	10
	10 x 20 cm	413711	3567385	EME047	10
	15 x 20 cm	413712	3567369	EME048	10
	20 x 25 cm	413713	3567393	EME049	10
	20 x 40 cm	413715	3567377	EME128	10



[1] Barrett, S. et al. (2018). A 50 patient observational clinical study of a superabsorbent dressing: effective exudate management with Zetuvit® Plus evaluated by clinicians and patients. Journal of Wound Care vol. 27, no. 2, February 2018

For more information contact your local HARTMANN representative or contact our customer services team

HARTMANN UK & Ireland +44 (0) 1706 363200
Ireland: +353 (0) 1800 937766
Email: info@uk.hartmann.info | www.hartmann.co.uk

Never miss an update from HARTMANN UKI
Join www.hartmannmarketing.com online now

HARTMANN UKI Healthcare Professionals

HARTMANN GROUP

©2020 Paul Hartmann Limited.
® represents a trademark of Paul Hartmann Limited,
a company registered in England and Wales
(company number 01523121) WM-00026/20-GB

Wound Management

Ask about our wider portfolio

Right Product | Right Wound | Right Time

