

# Zetuvit<sup>®</sup>

## Plus Silicone Border

Effective exudate management  
and excellent handling

### Open Labelled Non-Comparative Study

A multi-center observational study of 52 patients  
treated with an innovative self-adhesive siliconised  
superabsorbent dressing



# Zetuvit® Plus Silicone Border

– We are all in this together

For 200 years HARTMANN has worked together with Health Care Professionals (HCPs) to find the most effective, simple and convenient wound treatment solutions that benefit both HCPs and patients. One of the outcomes of the LINK Network is the **Zetuvit® Plus Silicone Border** – Open Labelled Non-Comparative Study.



LINK

Learn. Inform. Network. Knowledge.

Design:	Open Labelled Non-Comparative Study
Number of Patients:	52
Types of Wounds:	A variety of different types of moderately to highly exuding wounds [e.g. venous leg ulcers (40%), diabetic foot ulcers (29%), pressure ulcers (8%) and malignant wounds (8%)]
Location:	United Kingdom

## Objective:

Evaluate the effectiveness of **Zetuvit® Plus Silicone Border** in the management of moderately to highly exuding wounds.

## Effective exudate management:

**94 %**  
successful  
exudate  
management



**Effective** management of moderate to highly exuding wounds



Highly rated **prevention of exudate strikethrough\***



**1 of 3 patients**

**Improved** wound edge (29%) and peri-wound skin (36%) **within just 14 days**



**20 %**  
in 14 days

Significantly **reduced wound area**



**91 %**  
"Excellent"/  
"Good"

When used **under compression**

## Reference:

Barrett, S. et al. (2020). Treatment of 52 patients with a self-adhesive siliconised superabsorbent dressing: a multicentre observational study. Journal of Wound Care vol. 29, no. 3, June 2020.

\*Rated 78% excellent/good for prevention of exudate strikethrough




Almost  
**90 %**  
"Excellent"/  
"Good"

Handling  
and application

## Excellent handling and easy application:

- Zetuvit® Plus Silicone Border was rated "Excellent"/"Good" in:

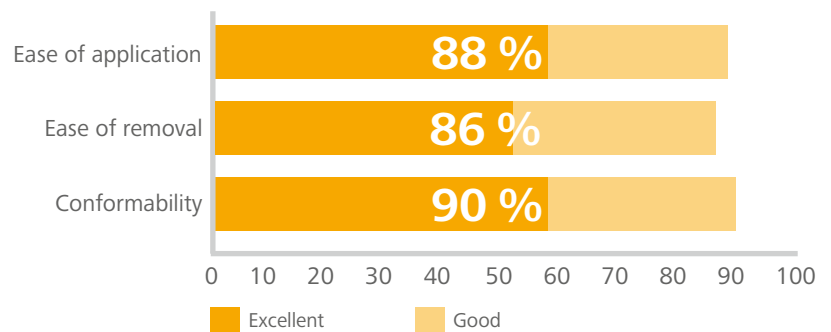

- The dressing remained fully in place in 86% of cases with high ratings for its ability to stay in place

## Improved Quality of Life:

- Enabled **longer wear times** thanks to successful exudate management
- Prevention of trauma and **painless dressing changes** thanks to silicone interface



For wearing comfort and  
general satisfaction

**76 %**  
"Excellent"/  
"Good"

"This is **the best dressing**  
I've ever had"

Patient Quotes

*"Extremely  
pleased with  
how the product  
stayed in  
place"*

*"I can get my  
shoes on for the  
first time in 12  
years"*

**80 %**  
 Clinicians would continue to use  
**Zetuvit®**  
**Plus Silicone**  
**Border**



Going further  
 for health

Product	Size/Wound Pad	Product Code	PIP Code	NCP Code	Pack Size
Zetuvit® Plus Silicone Border	10 × 10 cm / 5 × 5 cm	413910	4137071	EJA254	10
	12.5 × 12.5 cm / 7 × 7 cm	413920	4137055	EJA255	10
	17.5 × 17.5 cm / 11.5 × 11.5 cm	413930	4137063	EJA256	10
	15 × 25 cm / 9 × 19 cm	413940	4137048	EJA257	10
	20 × 25 cm / 14 × 19 cm	413950	4136030	EJA258	10
Zetuvit® Plus Silicone	8 × 8 cm / 6 × 6 cm	413810	4065074	EKH091	10
	12.5 × 12.5 cm / 10.5 × 10.5 cm	413820	4065082	EKH128	10
	10 × 20 cm / 8 × 18 cm	413830	4065090	EKH127	10
	20 × 20 cm / 18 × 18 cm	413840	4065108	EKH129	10
	20 × 25 cm / 18 × 23 cm	413850	4065116	EKH130	10



**Reference:**

- [1] Data on file: 27.Z+Sil-Border\_benchmark. [2] Data on file: Dressing Heat and Water Vapor Report 20.07.2018
- [3] Data on file: 27. Z+SilBorder\_ Add\_Feat\_barrier.

For more information contact your local HARTMANN representative or contact our customer services team

HARTMANN UK & Ireland +44 (0) 1706 363200  
 Email: [info@uk.hartmann.info](mailto:info@uk.hartmann.info) | [www.hartmann.co.uk](http://www.hartmann.co.uk)

Never miss an update from HARTMANN UKI  
 Join [www.hartmannmarketing.com](http://www.hartmannmarketing.com) online now

HARTMANN UKI Healthcare Professionals

HARTMANN GROUP

©2020 Paul Hartmann Limited.

® represents a trademark of Paul Hartmann Limited, a company registered in England and Wales (company number 01523121) WM-00084/20-GB

**Wound Management**  
 Ask about our wider portfolio  
 Right Product | Right Wound | Right Time

