

# Zetuvit<sup>®</sup> Plus Silicone

Highly effective exudate  
management and  
undisturbed wound healing

Open Labelled Non-Comparative Study  
Evaluation of a superabsorbent wound dressing,  
patient and clinician perspective: a case series

**Source:**

The Zetuvit<sup>®</sup> Plus Silicone Open Labelled  
Non-Comparative Study was published in the  
**Journal of Wound Care vol. 29, no. 3,**  
March 2020



# Zetuvit® Plus Silicone

– We are all in this together

For 200 years HARTMANN has worked together with Health Care Professionals (HCPs) to find the most effective, simple and convenient wound treatment solutions that benefit both HCPs and their patients. One of the outcomes of the LINK Network is the **Zetuvit® Plus Silicone** - Open Labelled Non-Comparative Study.





LINK

Learn. Inform. Network. Knowledge.

Design:	Open Labelled Non-Comparative Study
Number of Patients:	50
Types of Wounds:	A variety of different types of moderately to highly exuding wounds [e.g. venous leg ulcers (12.8%), diabetic foot ulcers (8.5 %)]
Location:	United Kingdom
<b>Objective:</b>	Evaluate the fluid management capabilities of the <b>Zetuvit® Plus Silicone</b> dressing and if it enables undisturbed wound healing.

## Highly effective exudate management:

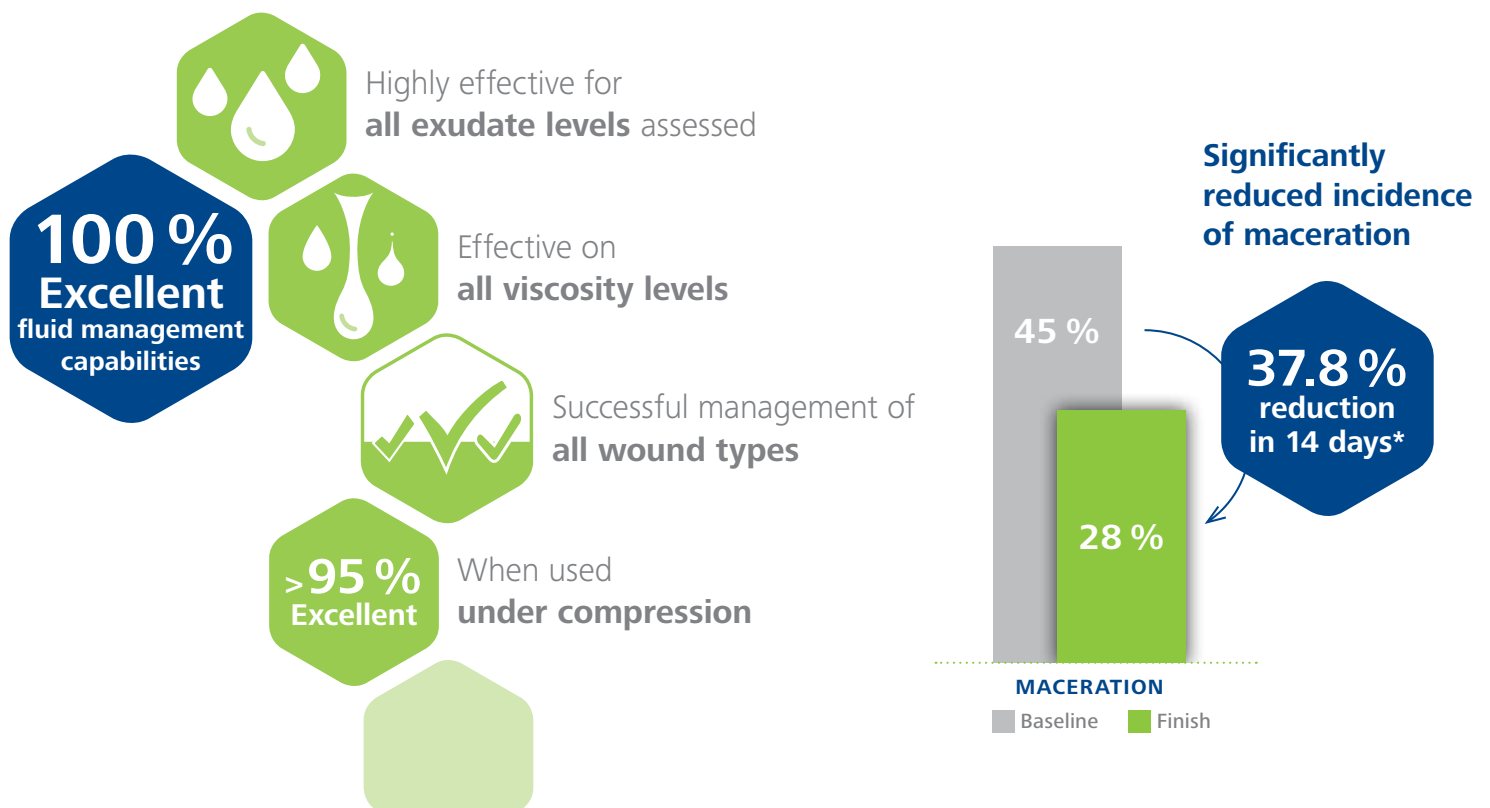

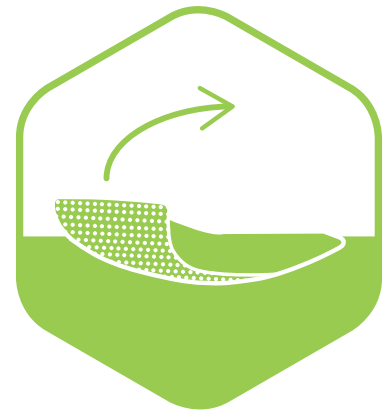

### Undisturbed wound healing:

Thanks to its silicone layer, Zetuvit® Plus Silicone doesn't adhere to the wound and prevents periwound damage

**100 %**  
clinicians  
agree

that it maintained an environment of undisturbed healing



### Improved Quality of Life:

- **Patient self-confidence** improved due to reduced odour, leakage or strikethrough
- **Painless and atraumatic** removal
- **98 % rating "Excellent" to "Good"** for both wearing comfort and general satisfaction

**100 %**  
"Good"-  
"Excellent"

**Flexibility and Conformability**



Clinicians would continue to use  
**Zetuvit®**  
Plus Silicone



Helps. Cares. Protects.

Product	Size/Wound Pad	Product Code	PIP Code	NHSCC Code	Pack Size
Zetuvit® Plus Silicone	8 × 8 cm / 6 × 6 cm	413810	4065074	EKH85003	10
	12.5 × 12.5 cm / 10.5 × 10.5 cm	413820	4065082	EKH85007	10
	10 × 20 cm / 8 × 18 cm	413830	4065090	EKH85006	10
	20 × 20 cm / 18 × 18 cm	413840	4065108	EKH85004	10
	20 × 25 cm / 18 × 23 cm	413850	4065116	EKH85005	10
Zetuvit® Plus Silicone Border	10 × 10 cm / 5 × 5 cm	413910	4137071	EJA254	10
	12.5 × 12.5 cm / 7 × 7 cm	413920	4137055	EJA255	10
	17.5 × 17.5 cm / 11.5 × 11.5 cm	413930	4137063	EJA256	10
	15 × 25 cm / 9 × 19 cm	413940	4137048	EJA257	10
	20 × 25 cm / 14 × 19 cm	413950	4137030	EJA258	10



**Reference:**

Atkin, L. et al. (2020). Evaluation of a superabsorbent wound dressing, patient and clinician perspective: a case series **Journal of Wound Care** vol. 29, no. 3, March 2020

For more information contact your local HARTMANN representative or contact our customer services team

HARTMANN UK & Ireland +44 (0) 1706 363200  
Ireland: +353 (0) 1800 937766  
Email: [info@uk.hartmann.info](mailto:info@uk.hartmann.info) | [www.hartmann.co.uk](http://www.hartmann.co.uk)

Never miss an update from HARTMANN UKI  
Join [www.hartmannmarketing.com](http://www.hartmannmarketing.com) online now

HARTMANN UKI Healthcare Professionals  
 HARTMANN GROUP

©2020 Paul Hartmann Limited.  
® represents a trademark of Paul Hartmann Limited, a company registered in England and Wales (company number 01523121) WM-000012/23-GB Feb 2023

**Wound Management**  
Ask about our wider portfolio  
Right Product | Right Wound | Right Time

