

HARTMANN



HydroTac[®] transparent
Transparent hydrogel dressing


SIMPLE AND EFFECTIVE

NEW

HydroTac® transparent

HydroTac® transparent comfort

HydroTac® transparent and transparent comfort are individually sealed, sterile wound dressings. Both versions release moisture and absorb a limited amount of exudate. The dressings promote wound healing by supporting the epithelialization and granulation process.^[1]

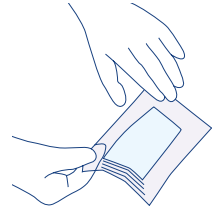

Support:

Provides the optimal moisture environment especially for low to non exuding wounds to support wound healing

Soothing:

Provides a soothing and cooling effect to the wound^[3]

Protective:

Dressing's outer layer acts as an effective barrier to bacterial penetration



Skin friendly:

Self-adhesive, with good skin tolerance

Transparent:

Transparent and moisturising dressing

When to use

- For dry or lightly exuding wounds during the granulation and epithelialisation stages
- Chronic and acute wounds – e.g. burns, abrasions, skin tears, etc.

Application

- Easy to handle
- Atraumatic dressing change^[1,2]
- Can remain on the wound up to seven days

HydroTac® transparent – available in 4 versions the non-border version (additional dressing material required for fixation)



5 x 7.5 cm



10 x 10 cm



10 x 20 cm



20 x 20 cm



HydroTac® transparent comfort – available in 4 versions features an adhesive border, coated with an acrylic based adhesive. Thus, no secondary dressing is required for fixation.



8 x 8 cm



12.5 x 12.5 cm



10 x 20 cm



6.5 x 10 cm



Product details

| Product | Product Sizes | | Product code | NHSSC code | PIP code | Pack size |
|---|----------------|----------------|--------------|------------|----------|-----------|
| | Overall | Wound pad size | | | | |
| HydroTac [®] transparent | 5 x 7.5 cm | n/a | 685905 | ELE124 | 4196481 | 10 |
| | 10 x 10 cm | n/a | 685906 | ELE120 | 4196515 | 10 |
| | 10 x 20 cm | n/a | 685907 | ELE121 | 4196507 | 10 |
| | 20 x 20 cm | n/a | 685908 | ELE123 | 4196499 | 10 |
| HydroTac [®] transparent comfort | 8 x 8 cm | 4 x 4 cm | 685925 | ELE119 | - | 10 |
| | 12.5 x 12.5 cm | 8 x 8 cm | 685926 | ELE122 | - | 10 |
| | 10 x 20 cm | 5.5 x 15.5 cm | 685927 | ELE125 | - | 10 |
| | 6.5 x 10 cm | 2.5 x 6 cm | 685928 | ELE128 | - | 10 |



[1] P. Zollner et al (2007) Clinical performance of a Hydrogel dressing in chronic wounds, a prospective observational study. Journal of Wound Care 16, No. 3, March 2007 [2] (2018) Clinical Evaluation Report, Hydrosorb[®] transparent and hydrosorb[®] transparent comfort. 1.0/03.07.2018 [3] Bazire et al (2015) Hydrosorb[®] versus control (water based spray) in the manner of radio-induced skin toxicity: Results of a multi-centre controlled random trial. Radiotherapy and Oncology 117, 229-23

For more information contact your local HARTMANN representative or contact our customer services team

HARTMANN UK & Ireland +44 (0)1706 363200 | Email: info@uk.hartmann.info | www.hartmann.co.uk

Never miss an update from HARTMANN UKI - Join www.hartmannmarketing.com online now

 HARTMANN UKI Healthcare Professionals  HARTMANN GROUP

©2023 Paul Hartmann Limited. ® represents a trademark of Paul Hartmann Limited, a company registered in England and Wales (company number 01523121) WM-000028/23-GB April 2023