

## Fitting guide


### Disposable underpads

# MoliCare® Premium Bed Mat

HARTMANN




### 4-step application for mobile people

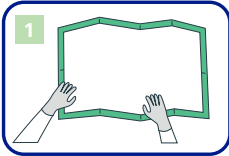

1 Unfold the bed mat completely. Place PE backed side facing the mattress, chair, wheelchair or other surface.



2 Position product lengthwise, crosswise or square bed mats diagonally as required.

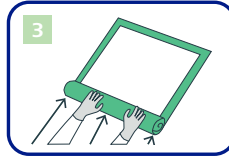


3 To remove the bed mat, roll or fold it inwards.




4 Dispose of the soiled bed mat in a suitable rubbish bin.


### 8-step application for bedbound people

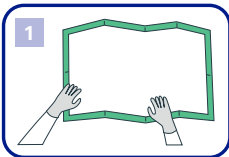


1 Unfold the bed mat completely.

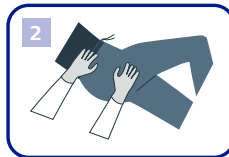


2 Safely turn person into recovery position.

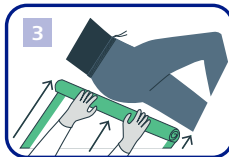

3 Place bed mat with PE backed side facing the mattress, parallel to the person with one third of the bed mat rolled in next to the buttocks.

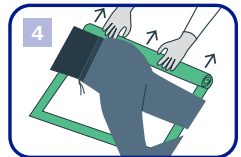


4 Turn the person over to their other side, then uncoil the remaining two thirds and return person into back position.

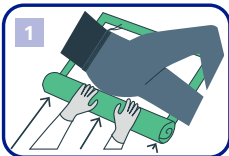

5 To remove the bed mat, securely turn person into recovery position. Roll up used bed mat towards the centre of the bed to the back of the person.



6 Place new bed mat mirrored – with the rolled-in side towards the person – next to the used one.

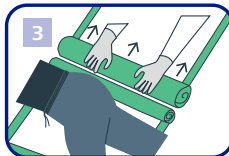


7 Turn person over their back onto their other side. Remove used bed mat by rolling it further inwards. Roll out replacement bed mat and return person into back position.




8 Dispose of the soiled bed mat in a suitable rubbish bin.

## Fitting for bed mat with wings

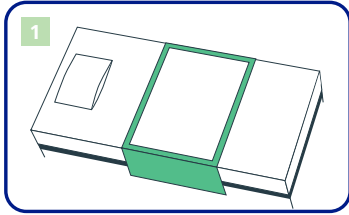


If using a MoliCare® Premium Bed Mat with wings, place the bed mat across the bed with PE side facing the mattress.

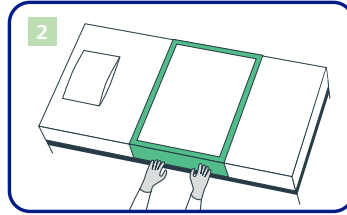

Firmly tuck wings under mattress on both sides to secure the bed mat.

## MoliCare® Premium Bed Mat

For moderate to severe incontinence

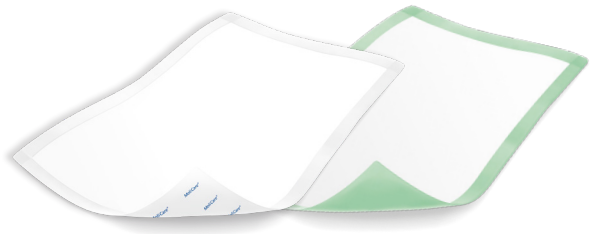

Absorbency	Code	Size	Working Capacity (ml)	Pack Size	Case Quantity
5	161 061	40 x 60cm	150	30	6
	161 063	60 x 60cm	250	30	4
	161 065	60 x 90cm	350	30	4
7	161 067	40 x 60cm	200	30	6
	161 068	50 x 60cm	350	25	4
	161 070	60 x 90xm	450	25	4
	161 072	60 x 90cm with wings	400	30	4
9	161 076	60 x 60cm	650	30	2
	161 078	60 x 90cm	950	30	2

To order additional products, go to [www.hartmanndirect.co.uk](http://www.hartmanndirect.co.uk) or call **0800 028 9490**