


HARTMANN

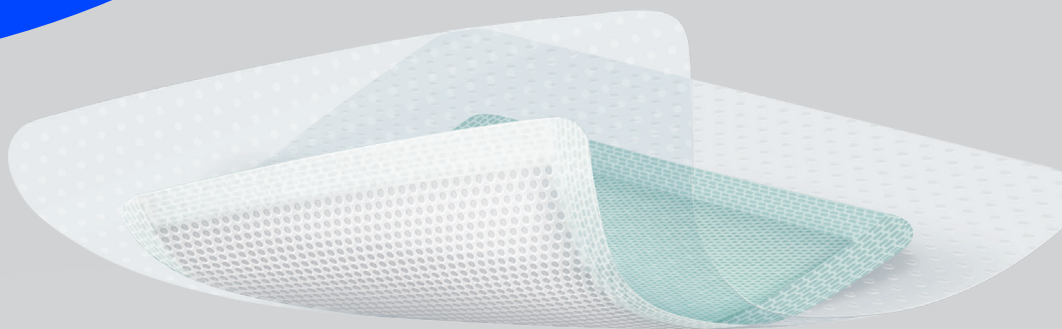

Patient Information Leaflet

Why Zetuvit® Plus Silicone Border?

Your wound has been dressed using Zetuvit® Plus Silicone Border. Zetuvit® Plus Silicone Border is a superabsorbent dressing with a silicone adhesive wound contact layer and adhesive border. It is suitable for a wide range of wounds. The dressing will absorb wound fluid and lock it away protecting your wound and surrounding skin.

Application and removal



Wash hands and clean the wound as instructed by your healthcare professional. Ensure the dressing exceeds the wound edges by approx. 1 to 2 cm.



Peel away the clear protective sheet from the back of the dressing. Place onto the wound, so that the green side faces away from the wound. Remove the remaining clear protective sheet.



Smooth down dressing to ensure the dressing is in contact with the wound and surrounding skin.

To remove – simply lift the edge and peel dressing away from the skin.

For further help and advice please watch our application video



When to change

Change your dressing as advised by your healthcare professional or if the wound fluid reaches the adhesive border.

Frequently asked questions



1. Which size should I apply?

Under the guidance of a healthcare professional, choose a dressing that exceeds the wound margins by approx. 1 to 2 cm.

2. Will it be painful when I change my dressing?

The dressing is composed of silicone adhesive, so it prevents the dressing from slipping and helps application. It also means that dressing changes can be carried out almost painlessly.

3. Can I wear this dressing in the shower?

Yes. These dressings provide a waterproof backing that allows for showering.

4. Can I cut the dressing?

You can cut the transparent adhesive border if required however you cannot cut the pad.

5. How can I get it?

Zetuvit® Plus Silicone Border is available on prescription.

Zetuvit® Plus Silicone Border should be used under the guidance of a healthcare professional. If the condition of your wound changes please contact them for further advice.

Size	Wound pad	Product code	Pack size	PIP code	NHSSC code
Zetuvit® Plus Silicone Border					
10 × 10 cm	5 × 5 cm	413910	10	4137071	EJA254
12.5 × 12.5 cm	7 × 7 cm	413920	10	4137055	EJA255
17.5 × 17.5 cm	11.5 × 11.5 cm	413930	10	4137063	EJA256
15 × 25 cm	9 × 19 cm	413940	10	4137048	EJA257
20 × 25 cm	14 × 19 cm	413950	10	4137030	EJA258



For more information contact your local HARTMANN representative
Email: info@uk.hartmann.info | www.hartmann.co.uk

HARTMANN UKI Healthcare Professionals HARTMANN GROUP

©2023 Paul Hartmann Limited. ® represents a trademark of Paul Hartmann Limited, a company registered in England and Wales (company number 01523121) WM-000052/23-GB July 2023

HARTMANN



Helps. Cares. Protects.