

HARTMANN



Hydrofilm® Application Guide



Hydrofilm® is a sterile film barrier that reduces the risk of infection and protects against physical damage on dry and primary healing wounds.

Application and removal



Clean the application area thoroughly and ensure that the skin is dry and free from creams and ointments. Hairy skin areas should be shaved before applying Hydrofilm®. Open the peel pouch and take out the dressing.



Hydrofilm® is protected by 4 covers, numbered 1 to 4 in the order of application. Slightly remove the protective covers 1 and 2. Place the adhesive side of Hydrofilm® on the wound and completely remove the protective covers 1 and 2. Gently press the the dressing onto the skin, conforming it to the body contours.



Remove the protective covers 3 and 4 and gently press Hydrofilm® to the skin.



Removal


Hydrofilm® can be removed almost painlessly by overstretching the material. To do this, loosen an edge and carefully take off the dressing by overstretching it.

Hydrofilm®



Size	Product code	Pack size	PIP code	NHSSC code
6 x 7 cm	970000	10	4326665	-
10 x 12.5 cm	970002	10	3426228	-
10 x 15cm	970004	10	2667350	-
15 x 20cm	970006	10	3426244	-
10 x 25cm	970008	25	3426236	-
12 x 25cm	970009	25	3427368	-
20 x 30cm	970010	10	3426351	ELW248

For further help and advice please watch our application video



For more information contact your local HARTMANN representative
 Email: info@uk.hartmann.info | www.hartmann.co.uk

 HARTMANN UKI Healthcare Professionals  HARTMANN GROUP

©2023 Paul Hartmann Limited. ® represents a trademark of Paul Hartmann Limited, a company registered in England and Wales (company number 01523121) WM-000093/23-GB November 2023



Helps. Cares. Protects.